

Use of Face Shields



Face Shields are useful in further reducing exposure to others.

Face Masks are *Required*.

Face Shields are *Optional*.



Cleaning Your Face Shield

Although these face shields are labeled as “disposable,” they can be reused several times (which will vary). They can continue to be used as long as they remain intact. Below are some guidelines for cleaning plastic face shields:

1. While wearing gloves, carefully wipe the inside, followed by the outside of the face shield using a clean cloth saturated with neutral detergent solution or cleaning wipe.
2. Carefully wipe the outside of the face shield using a wipe or clean cloth saturated with EPA-registered disinfectant solution.
3. Wipe the outside of face shield with clean water or alcohol to remove residue.
4. Fully dry (air dry or use clean absorbent towels).
5. Remove gloves and perform hand hygiene.